

USER'S MANUAL

ATOZ Massage chair

Dr.Care®



www.drcare.us



Made By
Dr.Care International Inc.
6565 E. Washington Blvd., Commerce,
California, 90040, The U.S.A
Tel: +1 (229) 800.0009
www.drcare.us

CONTENTS

Introduction:.....1

Safety Introduction:.....2

Product specifications:.....5

Warnings:.....6

Preliminary checks:.....8

Parts:.....11

Functions:.....12

Remote control instructions:.....14

Usage methods:.....17

Illustration:.....38

Maintenance History:.....40

Contents:.....44

INTRODUCTION

Thank you for choosing Dr.Care`s products!

*The **ATOZ massage chair** is a high technology item to improve your health, to relax your body after a hard working day. There will be no more stress, any aches or pains, ... The **ATOZ massage chair** will bring you more for a happy life, an optimism, sound mind and sound body.*

With over 24 years experience in the field of researching and manufacturing health care products, Dr.Care are non-stop researching and developing to create the modern unique leading products, more and more professional massage technique to satisfy Dr.Care`s customers.

To achieve the highest efficiency in treating pains and health recovery, please carefully read this User Manual book.

We always look forward to serving you better.

Your sincerely,



Brian Williams

SAFETY INTRODUCTION

1.Important Safety Warnings

- Do not allow children to touch any movable part of this product.
- Use the well-grounded power supply unit suitable for this product.
- Pull out the plug after use or before cleaning to avoid any physical injury or damage to the product.
- Operate this product as instructed in this manual.
- Do not use any parts or accessories which are not recommended.
- Do not use this product outdoors.
- Please read the Operation Instructions carefully before operation.
- Never use this product for any other purpose not listed herein.
- 20 minutes of usage each time is recommended.
- Do not use this product if the leather is broken or damaged.
- Do not use this product if the cover or leather is broken or damaged.
- Do not use this product if the radiator hole is covered or blocked.
- Do not drop anything into this product.
- Never fall asleep while using this product.
- Do not use this product if you are drunk or feel unwell.
- Do not use this product within one hour after a meal.
- Do not use this product with excessive force to avoid any injury.
- Do not carry any fragile, explosive, or combustible material in your pocket, such as a mobile phone and lighter when using the massage chair.

2. Operating Environment

- Do not use this product in a very moist environment such as the bathroom.
- Stop using this product immediately when the ambient temperature changes sharply.
- Do not use this product in any heavily-dusty or caustic environment.
- Do not use this product in a small space or a place without good ventilation.
- The normal room temperature ranges from 0 °C to 35 °C.
- Do not use this product in an excessively humid room such as a bathroom. Use it at a relative humidity between 20% and 80%.
- Do not use this product outdoors.
- Avoid direct sunlight.
- Keep this product away from any heat source such as a stove and air outlet of an air conditioner.
- Do not use this product during thunder or lightning weather.
- Do not use this product in dusty places.

MAINTENANCE HISTORY

Date:
Problems:
Solution:
Signed by Technician	
Date:
Problems:
Solution:
Signed by Technician	
Date:
Problems:
Solution:
Signed by Technician	

MAINTENANCE HISTORY

SAFETY INTRODUCTION

<p>Date:</p> <p>Problems:</p> <p>Solution: Signed by Technician</p>
<p>Date:</p> <p>Problems:</p> <p>Solution: Signed by Technician</p>
<p>Date:</p> <p>Problems:</p> <p>Solution: Signed by Technician</p>

3. The following people should not use this product

- Patients with osteoporosis.
- Patients with heart disease or wearing electronic medical devices such as a pacemaker.
- People who have a fever.
- Pregnant women or women on their menstrual period.
- Patients who are injured or have a skin disease.
- When unsupervised, any children are under the age of 14 and those unconscious are not allowed to use this product.
- People who are told by their physicians to have rest or who feel unwell.
- People whose bodies are wet are forbidden to use this product.
- This product has a hot surface. Persons insensitive to heat must pay attention when using this product.

4. Safety Precautions

- Check whether voltage is consistent with the specification of this product.
- Never insert or pull out the plug with a wet hand.
- Do not let water flow into this product to avoid electric shock or cause damage to this product.
- Do not pull the power cord when plugging or unplugging it. Rough handling is forbidden.
- Do not damage the wire or modify the circuit of this product.
- Do not clean live parts of this product with a wet cloth such as a power switch and plug.
- Be away from this product in case of power failure to avoid any injury if power is restored suddenly.
- Stop using this product at once while it works abnormal, and consult your local supplier.
- Stop using this product if you feel unwell, and consult your health care practitioner.
- This product is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning the use of this product by the personnel responsible for their safety. Children should be supervised to ensure that they do not play with this product.
- If the power cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified staff in order to avoid hazards.

SAFETY INTRODUCTION

5. Product Service and Maintenance

- This product should be maintained by the service center designated by the manufacturer only. No user should disassemble or maintain it without permission.
- Do not forget to turn off the main power switch after usage.
- Do not use this product if the power socket becomes loose.
- If this product will be left unused for a long period of time, it is necessary to roll up the power cord of this product and store this product in a dry and dust-free environment.
- Do not store this product at a high temperature or near open flame. Avoid prolonged exposure to direct sunshine.
- Please clean this product with dry cloth. Never use thinner, benzene or alcohol.
- The mechanical components of this product are specially designed and manufactured, so no special maintenance is needed.
- Do not use a sharp object to stab at this product.
- Do not roll or pull this product on any uneven ground. Instead, it should be lifted before moving.
- Please use this product intermittently. Do not keep it working continuously for a long period of time.

6. Solutions to Ordinary Malfunctions

- If the motor sounds during operation of this product, this is the normal operation sound.
- If the controller cannot work normally, check whether the plug and the power socket are connected reliably, and whether the power switch is turned on.
- If the rated working time is over, this product will automatically activate the power-off button; If this product keeps operating for a long period of time, the temperature protector will stop this product automatically. The product can be used again after half an hour.

7. Storage Conditions

- Storage room temperature: -5 °C to 35 °C.
- Relative humidity (RH) between 20% and 80%; free of corrosive gases, and a well-ventilated room.

MAINTENANCE HISTORY

Date:	Problems:	Solution: Signed by Technician
Date:	Problems:	Solution: Signed by Technician
Date:	Problems:	Solution: Signed by Technician

MAINTENANCE HISTORY

Date:

Problems:

Solution:
Signed by Technician

Date:

Problems:

Solution:
Signed by Technician

Date:

Problems:

Solution:
Signed by Technician

PRODUCT SPECIFICATIONS

ATOZ massage chair

PRODUCT NAME : ATOZ massage chair
MODEL : DR-AZ 849S
RATED VOLTAGE : 120V~
RATED INPUT : 150W 1.25A
RATED FREQUENCY : 60Hz
SAFETY DESIGN : CLASS I
SERIAL NUMBER :
 DR-AZ 849S

Dr.Care
better a little each day




Made by
Dr.Care International Inc.
 6565 E. Washington Blvd., Commerce,
 California, 90040, The U.S.A
www.drcare.us

WARNINGS


! Warnings

When the legrest of the massage chair is lifted, nobody should apply pressure on the legrest to prevent the massage chair from falling forward which will damage this product or result in personal injuries.

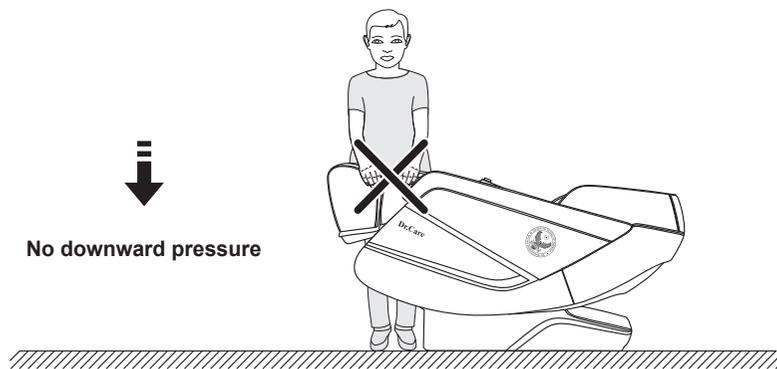

! Warnings


Before sitting on the massage chair, make sure that the massage chair manipulator does not stay in the seat cushion position to avoid discomfort to the buttocks during the body pressing down and any damage to the massage chair manipulator.

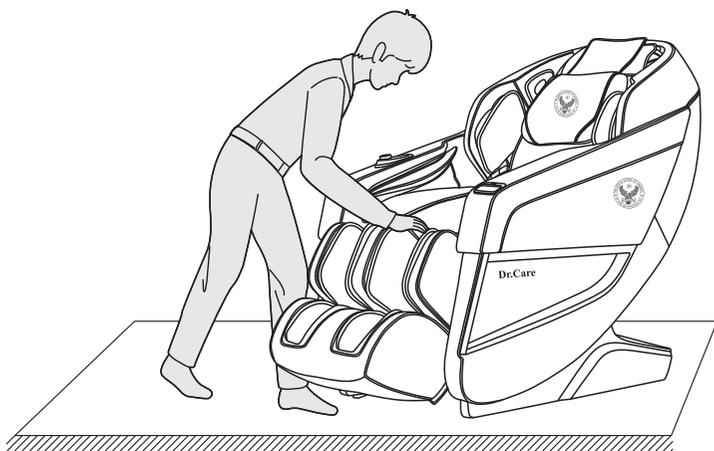

ILLUSTRATION

Acupressure Point Massage Instruction

Every Vertebra is related with the health, for example:

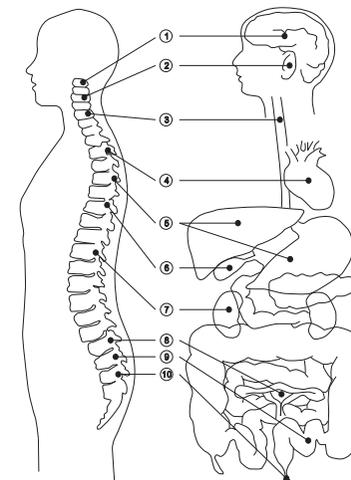


- | | |
|---|---|
| 1. The first bone of neck vertebra diagnoses headache. | 6. The ninth bone of chest vertebar diagnoses chololith. |
| 2. The third bone of neck vertebra diagnoses hard of hearing. | 7. The sixth bone of chest vertebra diagnoses nephropathy. |
| 3. The fifth bone of neck vertebra diagnoses bronchus. | 8. The third bone of waist vertebra diagnoses constipation and back pain. |
| 4. The second bone of chest vertebra diagnoses cardiopathy. | 9. The fifth bone of waist vertebra diagnoses hemorroid. |
| 5. The fourth bone of chest vertebra diagnoses liver disease and tummy bug. | 10. Hemorrhoids, Anus, Enhance Physiology. |

ILLUSTRATION

Zone Therapy Charts By Symptoms


Every Vertebra is related with the health, for example:

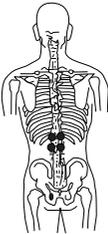Headache-Insomnia 	Shoulder 	Nerve Problem 	Knee Pain 
Heart Problem 	Diabetes 	Liver Problem 	Kidney Problem 
Rheumatism 	Arthritis 	Back Pain 	Sciatica 

WARNINGS


⚠ Warnings

Do not remove or pull out the remote controller cable or the power cord in the standby or power-on state to avoid any damage to this product or accidental electric shock of a body part. You are advised to unplug the remote controller cable and the power cord and put them away after you use the massage chair.

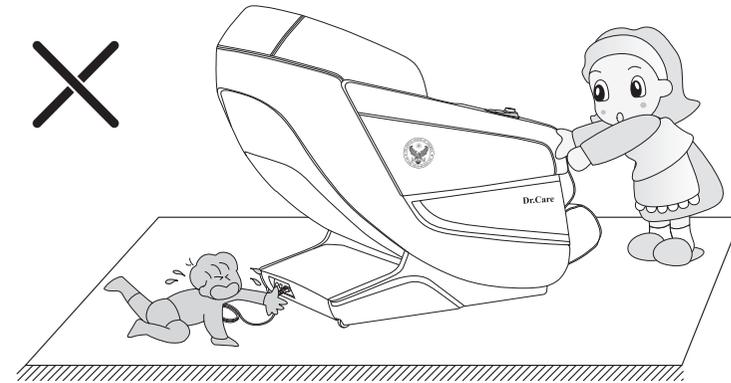

⚠ Warnings


When the zero-gravity lying function is enabled, be sure to check whether there is any child or pet staying or playing under the bottom of the legrest and backrest, in order to avoid any damage to this product or physical injury.

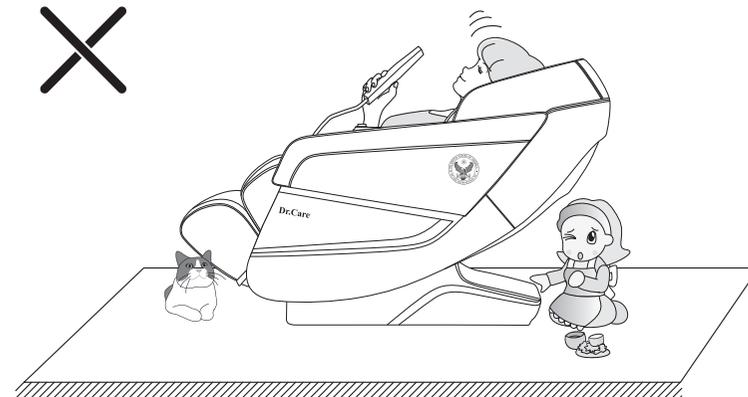

PRELIMINARY CHECKS

I. Turn off power and stop massage

- During the massage process, press the power switch key to immediately turn off all massage functions, the seat back and the foot pad are reset, the massage time is set up.
- If the power is suddenly shutdown, the chair can not be reset. Turn on the power.
- Cut off the power supply of the whole machine, the figure (schematic diagram of the whole machine power cut).

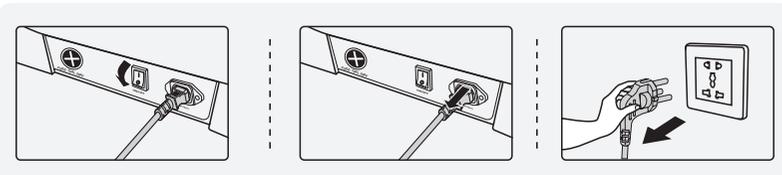


- 1 Schematic diagram of the whole machine power turning off.
- 2 Schematic diagram of unplugging the whole machine wiring.
- 3 Schematic diagram of unplugging the power plug.

II . Operation Instructions for Bluetooth Speaker Play

1. Enter the smart device manual and open the Bluetooth function to search the new Bluetooth devices.
2. Will find the bluetooth named like (DR-AZ849S-XXXXX), choose it to pair with the chair .
3. Turn on the Music Player in the smart device.

III. Installation Instructions for Legrest

- After connecting the wires and the air hose between the legrest and the massage chair, remove the cable ties and insert the connecting part into the seat (Figure 1).
- Place the round tubes on the left and right sides of the legrest onto the legrest support (Figure 2).

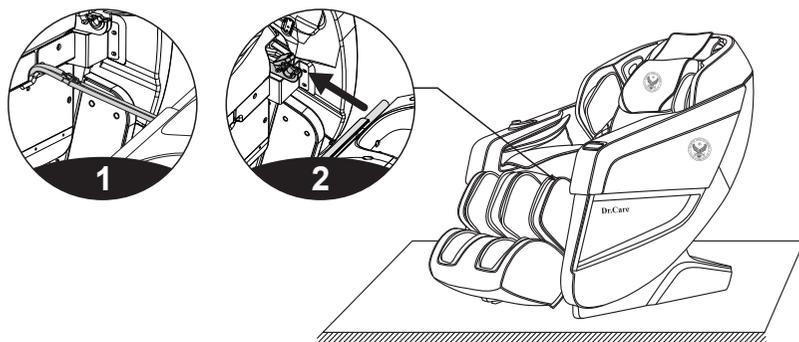

USAGE METHODS

24) Voice Control Mode

- When the massage chair is turned on or in standby mode, the sound “Hello” or “Xin chào” will be produced to enable the voice control function. Alternatively, you can press the voice control button on the right armrest to enable the voice control function.
- Long press 3 seconds to turn on or off the voice control function.
- After such function is enabled successfully, the massage chair will reply “Chào bạn” At this moment, once a corresponding voice control entry to be implemented is given within six seconds, the corresponding voice control function can be performed after a reply is received from the massage chair; if no correct voice control entry is given within six seconds, the voice control function automatically will be off automatically and needs to be enabled again.
- Note: When waking up the voice control function, please use Vietnamese, the speech rate is moderate, Voice commands should be follow up the Voice entry.

Voice Control Entry

Entry: “Xin chào!”. Chair’s Answer: Chào bạn.

No.	Voice control entry	Chair’s Answer
1	Khởi động	Xin chào quý khách!
2	Tắt máy	OK, tắt máy.
3	Nhẹ nhàng	OK, nhẹ nhàng.
4	Thư giãn	OK, thư giãn.
5	Toàn thân	OK, toàn thân.
6	Cổ vai	OK, cổ vai.
7	Lưng eo	OK, lưng eo.
8	Kéo giãn	OK, kéo giãn.
9	Cố định	OK, cố định.
10	Chế độ khác	OK, chế độ khác.
11	Xuống dưới	OK, xuống dưới.
12	Lên trên	OK, lên trên.
13	Túi khí	OK, túi khí.
14	Tắt túi khí	OK, tắt túi khí.
15	Nâng lên	OK, nâng lên.
16	Hạ xuống	OK, hạ xuống.


Remark: Command 11 and 12 can only be used in state of fixed position massage. They can use after command 9 or 10.

USAGE METHODS

Function	Description
	Power button: Short press this button to turn ON/OFF the massage.
	Turn key knob: By rotating this knob clockwise, the backrest will move downward slowly and the legrest will rise slowly; by releasing it, such DOWN/UP action of the backrest and the legrest will be stopped.
	Turn key knob: By rotating this knob counterclockwise, the backrest will rise slowly and the legrest will move downward slowly; by releasing it, such UP/DOWN action of the backrest and the legrest will be stopped.
	Press this button during massage, and all the massage actions will be suspended at once. (i.e. the massage chair will turn off after 20min). Then press this button again to restore all the previous massage actions.
	Automatic mode key: Press it to switch among those automatic massage functions.
	Voice control button: Turn ON/OFF the voice control function. (Turn ON / Speak mo giong noi)(Turn OFF / Speak tat giong noi)
	Heat treatment key: Click the key to enable or disable the heat treatment function.
	Air pressure key: upper air, lower air, full body air, and turn on/off air pressure massage function.
	Legrest up button: Pull the button to make the legrest rise slowly. Release it to stop such action.
	Legrest down button: Press the button to make the legrest move downward slowly. Release it to stop such action.
	Zero-gravity key I : the positions of the legrest and the backrest can be automatically adjusted to the positions I for zero gravity.
	Zero-gravity key II : the positions of the legrest and the backrest can be automatically adjusted to the positions II for zero gravity.
	Backrest up button: the back rest keeping lifting when pressing this button, and black rest stopped lifting once release the button.
	Backrest down button: the back rest keeping reclining when pressing this button, and black rest stopped reclining once release the button.


PRELIMINARY CHECKS

- Next, turn the legrest fixing cover over to cover the round tubes on both sides of the legrest (Figure 3).
- Fasten the legrest to the massage chair with two M4 screws (one on each side). Check whether there is any abnormality during turning up and down, or any tube, cable or wire is folded. If everything is OK, the legrest is installed (Figure 4).

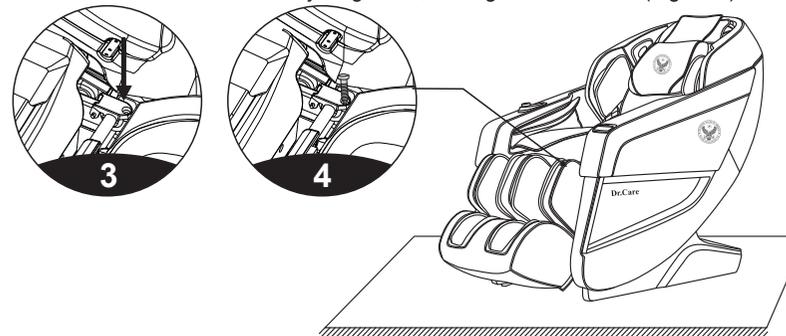

IV. Instructions for Removal and Use of Head Cushion and Backrest


- The use of the head cushion can reduce the intensity of kneading massage on the neck and shoulders. You can decide whether to use the head cushion according to your own needs (the head cushion is recommended). The backrest and the back cushion are connected by a zipper (1). The head cushion and the backrest are connected by a zipper (2).

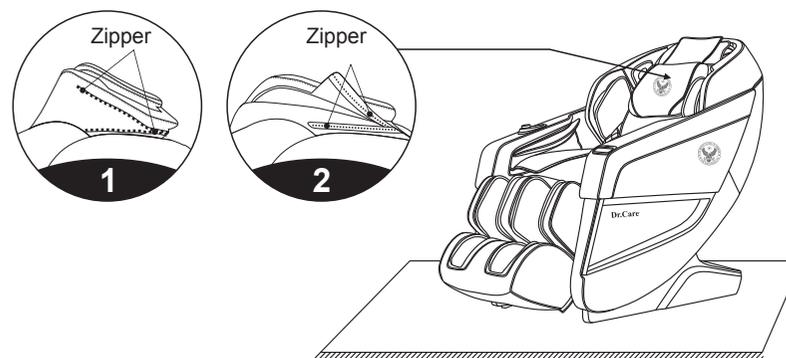

PRELIMINARY CHECKS

V . Floor Protection


- The prolonged heavy pressure on the massage chair may cause scratches on the floor. Thus, put a carpet or cushion on the floor where the massage chair is placed to avoid this damage.

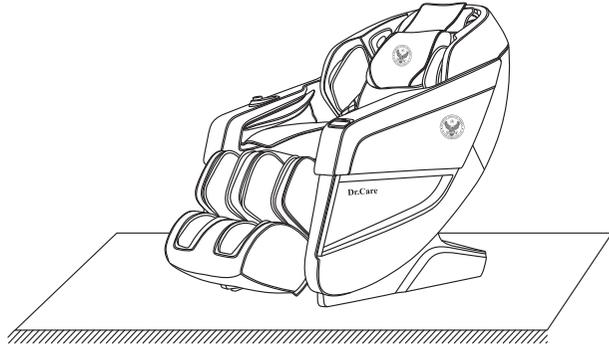

Note: Do not move the massage chair with its wheels on the rough ground or in a narrow space. At least two persons are required to lift it. During lifting, take the legrest sealing plate and backrest cover grooves as the acting points. (Do not lift the armrest directly.)

VI . Instruction for Movement


- Make sure that all the wires are pulled away from the massage chair surface; then press the backrest down to a certain angle (with the center of gravity resting on the wheels), you can push the massage chair forward with hands, move it to a suitable position and gently restore it to the normal angle. (Note: During movement, you must turn off the power button first and then unplug the power cord and the remote controller connection wire.)

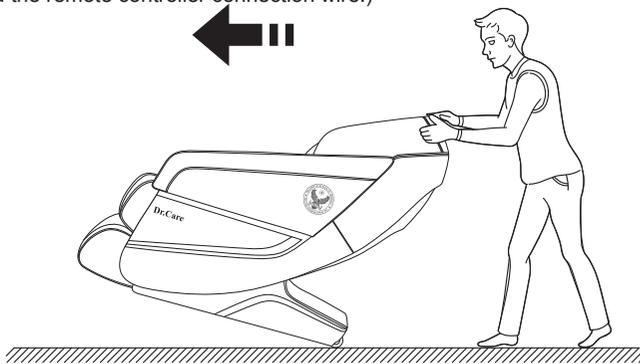

Warning : The rear moving wheels of the massage chair are only suitable for short-distance and slow movement. For long-distance transportation, please use other auxiliary moving tools to move this massage chair!

USAGE METHODS

23) Introduction to Armrest Shortcut Keys

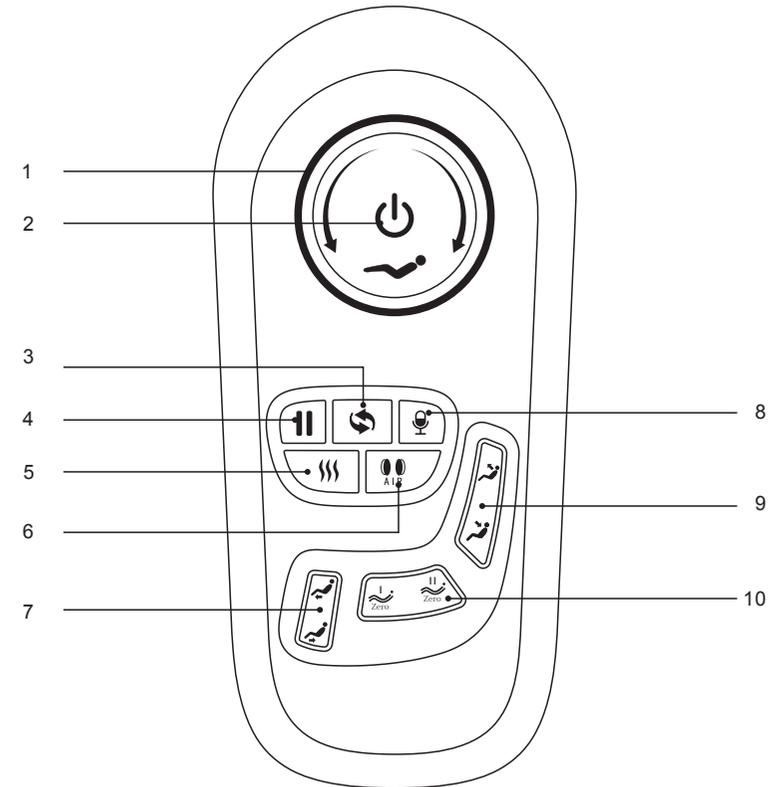


1. Turn key knob: Press it to perform linkage adjustment on the backrest for upward/downward movement.
2. ON/OFF key: Press it to start the massage chair.
3. AUTO key: Press it to enable the automatic massage function.
4. Pause/run button: Pause or run all massage functions.
5. Heat treatment key: Click the key to enable or disable the heat treatment function.
6. Air pressure key: Turn on/off air pressure massage function.
7. Legrest up/down key: Press it to make the legrest move upward/downward.
8. Voice Control Wake-up key: Short press this key to wake up the voice control, long press for 3 seconds to turn on/off the voice control function.
9. Backrest up/down key: Press it to make the backrest move upward/downward.
10. Zero-gravity key: Press it to adjust the massage chair to the angle for zero gravity.

USAGE METHODS


22) Introduction to armrest shortcut keys

Left armrest shortcut keys

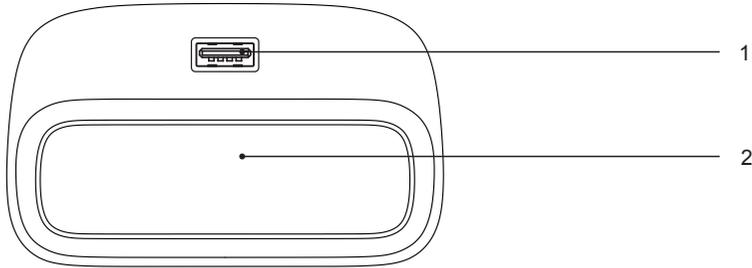

- 1.USB charging port: intelligent device charging.
- 2.Placement slot : Remote controller or mobile phone placement slot

PARTS

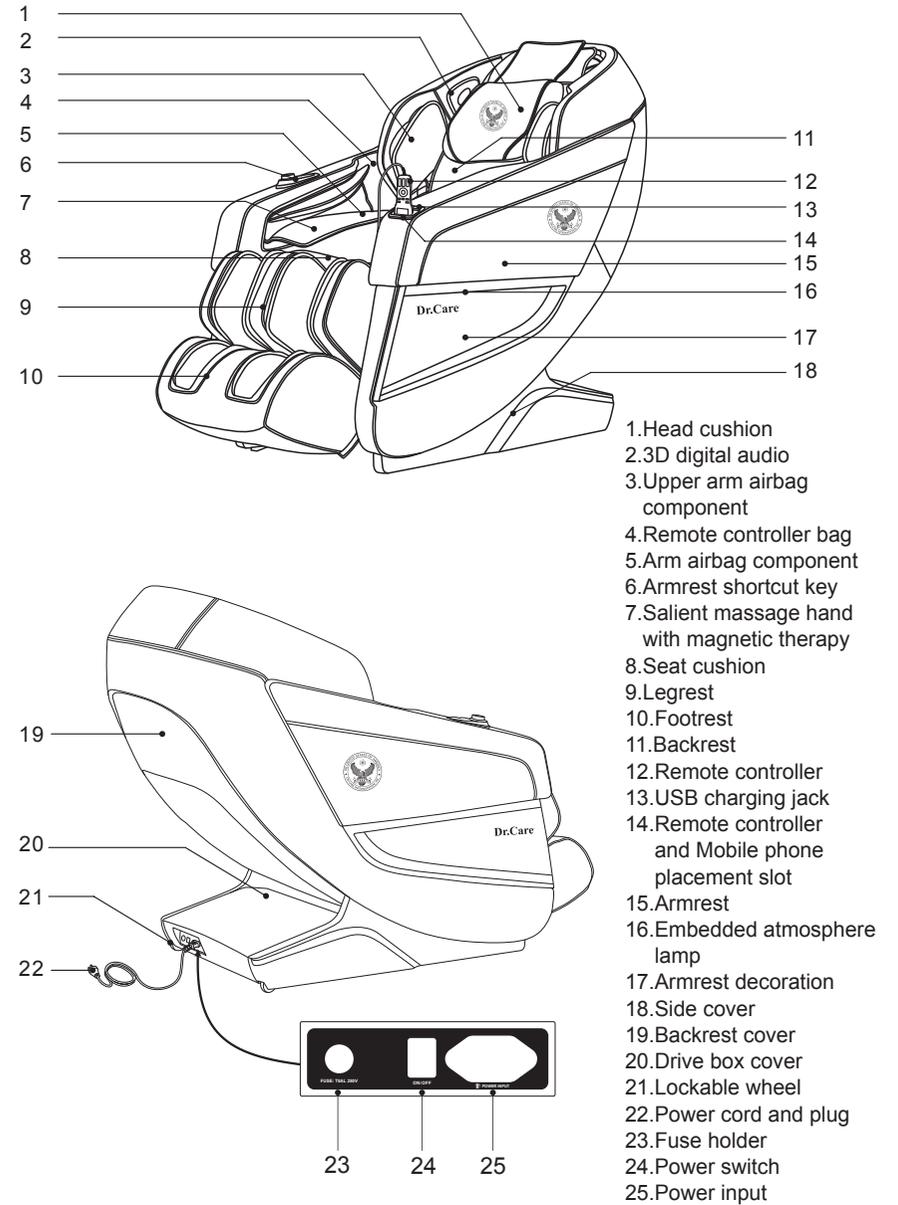

FUNCTIONS

This product is designed with a set of massage chair manipulators that can move up and down, drive four wheels and operate in mute mode.

- There are five simulative massage techniques: Shiatsu, Kneading, Tapping, Knocking , and simultaneous kneading and Tapping.
- This product is equipped with automatic shoulder detection and positioning.
- With 12 special automatic massage functions: A1-Stroke prevent, A2-Comfort, A3-Relax, A4-Full body, A5-Neck and shoulder, A6-Stretch, A7-Back and waist, A8-Chiropractic, A9-Fatigue recovery, A10-Meridians, A11-Beautiful buttock, A12-Full airbag
- There are five intelligent memory functions (M1, M2, M3, M4, M5): Remarks on saved massage functions and techniques are available (e.g. Dad, Mom, Son, Daughter, or none).
- This product is equipped with the manual massage function for the upper body: there are five kinds of massage position selections such as partial, fixed point, full, upper back and lower back; the speed of each massage manipulation can be adjusted with five speed levels. In the fixed point and partial mode, the manipulators can adjust the position up and down; during tapping, shiatsu and knocking massage, the width of the massage balls is selectable, i.e. wide, medium or narrow).
- Airbag massage function: Upper air and Lower air. Upper air include shoulder airbag massage and arm airbag massage; Lower air include calf and foot airbag massage. There is three levels of intensity for selection.
- This product is equipped with the back heat treatment function: carbon fiber is used as far infrared heat source.

USAGE METHODS

Note:

- According to the remaining massage time, when the displayed time is not enough for the addition and subtraction of a certain time, the operation will not be displayed (the upper limit of the time display is 40 minutes). For example: If the display time of the remote controller is 39 minutes, the massage time adjustment range will be: -5min, -10min (+10min or +5min cannot be displayed).
- The adjustment of airbag intensity can only be used when the airbag function or the automatic function is enabled (for more information about how to enable the airbag function, refer to Page 16 of the Operation Instructions).


21) Lying angle adjustment key

Button	Description
	Legrest up key: Hold down the button to make the leg rest rise slowly. Release it to stop such action.
	Legrest down key: Hold down the button to make the leg rest move downward slowly. Release it to stop such action.
	Linkage sitting key: By holding down this key, the backrest will rise slowly and the leg rest will move downward slowly; by releasing it, such UP/DOWN action of the backrest and the leg rest will be stopped.
	Linkage lying key: By holding down this key, the backrest will move downward slowly and the leg rest will rise slowly; by releasing it, such DOWN/UP action of the backrest and the Leg rest will be stopped.
	Zero gravity key: Automatically adjust the position of calf rest and back rest to zero gravity position each time you press it. 3 Zero gravity modes and OFF lying mode, cycling switch.
	Reset key: Press this key to reset the massage chair to its initial state.

USAGE METHODS

20) Massage adjustment

- In the state of the information interface, press "Left Button" or "Right Button" to select the massage adjustment button, and press "OK" to enter the massage adjustment interface.
- Massage adjustment includes: 6 massage adjustment functions can be selected, including back massage speed, back massage range, massage width, airbag intensity, roller speed, and massage time.
- After entering the massage adjustment interface, press "Up Key" or "Down Key" to select the desired massage adjustment functions, use "Right Button" to enter the massage level selection interface. Then press "OK" to confirm the selected level. Press "Left Button" to return to the previous menu interface.

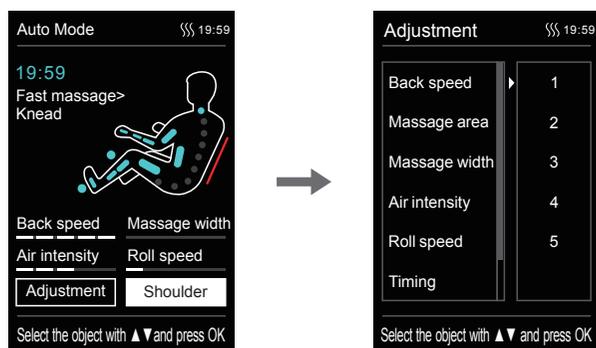


Function	Description
Back massage speed	5 speed levels adjustable
Massage range	There are 5 modes for your choice: spot, partial, full back, upper back, and lower back
Massage width	3 width levels adjustable
Airbag intensity	3 intensity levels adjustable
Roller speed	Choose from 4 modes including OFF and Level 1-3
Massage time	Choose from 4 modes: +10min, +5min, -5min, -10min

FUNCTIONS

- This product is equipped with the intelligent voice control and recognition function.
- In the airbags for the left and right palms, there are salient massage bands with magnetic therapy function.
- The slide rail technology for the backrest ensures zero space between the backrest and wall, so as to save the space to the greatest extent.
- The backrest guide adopts the curve guide technology, so it completely fits the curve of human back. The full body can enjoy full massage from the head to the thighs.
- The music player system is available. 3D digital audio systems are built into left and right upper arms. The Bluetooth function of the smart device can be used to play music wirelessly.
- At the calf have kneading massage function which can relieve the stiffness and stress of calf.
- The legrest is equipped with a stepless flex adjusting function suitable for users with different heights.
- Equipped with independent up/down adjustment function of the legrest and backrest, linkage up/down of legrest and backrest adjustment function, and zero-gravity function.
- The right armrest is provided with on/off key, linkage adjustment key, pause/run button, AUTO key, voice control button, heat treatment key, air pressure key, Legrest up/down key, zero gravity key, backrest up/down key.
- This product is equipped with the foot roller massage function.
- The left armrest is equipped with USB charging function. The armrests on both sides are equipped with LED atmosphere lamps.
- Equipped with TFT color screen manual remote controller. After shut down the chair, it will automatic sitting up, and the manipulator will reset as well.


REMOTE CONTROL INSTRUCTIONS

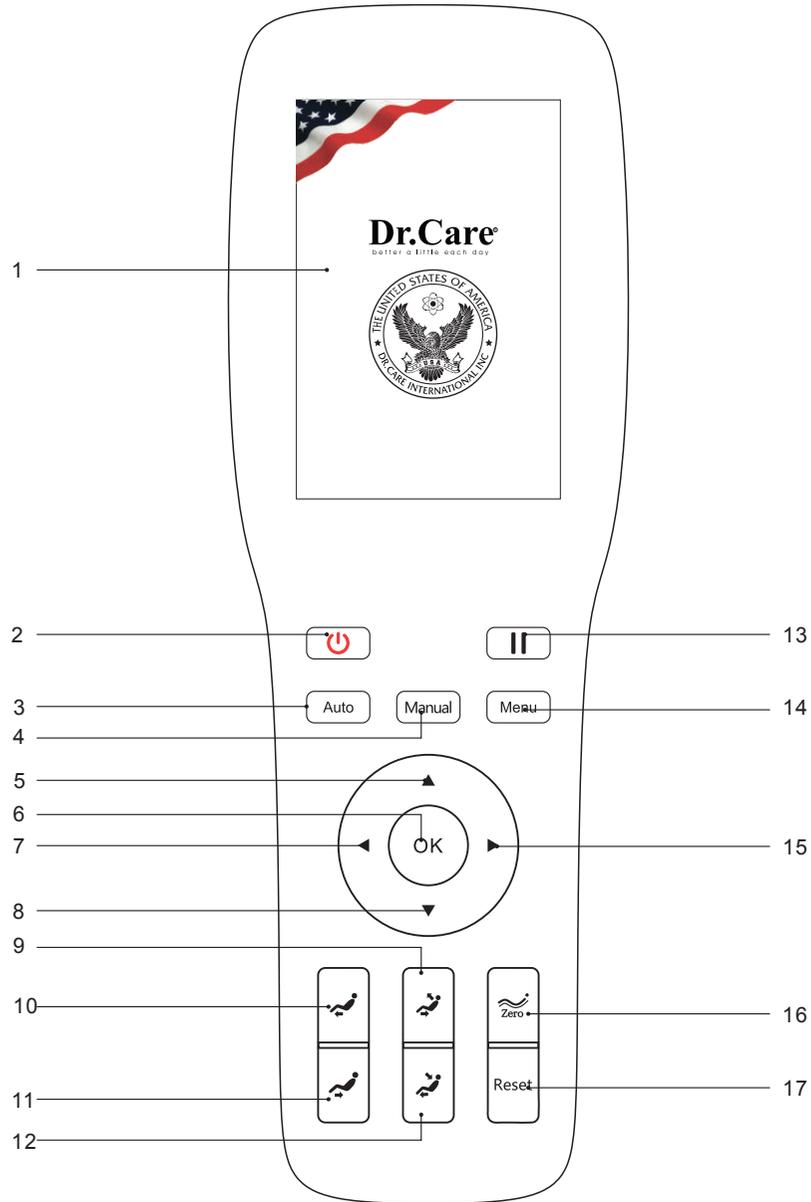

USAGE METHODS


18) Settings - Bluetooth settings

- Bluetooth settings (ON by default): After the Bluetooth function of the massage chair is enabled, the Bluetooth-enabled audio source device (e.g. mobile phone, MID tablet) can be connected with the Bluetooth module of the massage chair, and the music played by the audio source device can be wirelessly transmitted to the audio system of the massage chair via Bluetooth for music playback.
- Enter the settings interface. Press "Up Key" or "Down Key" to select the Bluetooth settings. Press "Right Button" to enter the Bluetooth ON/OFF interface. Press "OK" to select the required ON or OFF; press "Left Button" to return to the previous menu interface.

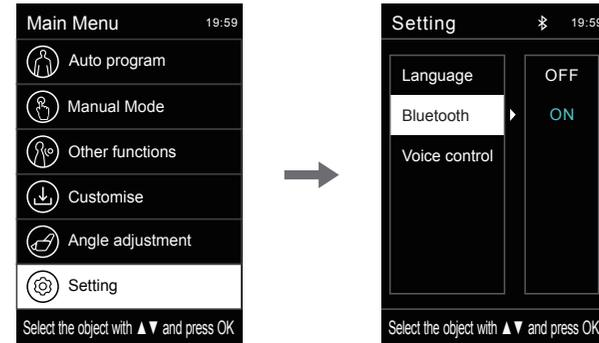

19) Intelligent Voice Control


- Intelligent voice control: enable or disable the intelligent voice control function (the function has a memory, that is, after such selection, power-off and restart, it will be the last selected state); Note: When the intelligent voice control is disabled, voice wakeup cannot be performed.
- Enter the settings interface. Press "Up Key" or "Down Key" to select the intelligent voice control function. Press "Right Button" to enter the intelligent voice control function settings. Press "OK" to select the required ON or OFF; press "Left Button" to return to the previous menu interface.

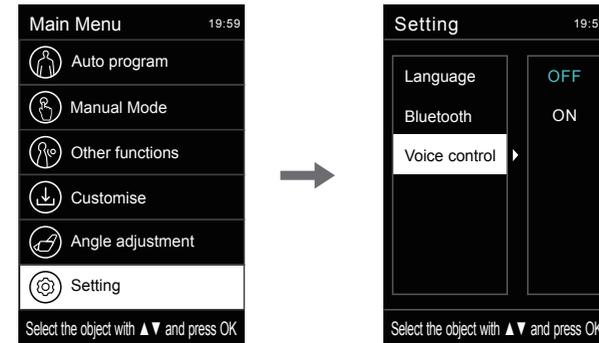

USAGE METHODS


16) Settings (Main Menu-Settings)

- Press "Menu Button" to enter the main menu interface. Press "Up Key" or "Down Key" to select the settings icon. Press "OK" to enter the settings interface.
- Settings include: language settings, Bluetooth settings, intelligent voice control, and other settings functions.
- After entering the settings interface, press the "Up Key" or "Down Key" to select the desired settings function. Select any one of the settings functions, and then press "Right Button" to set the selected settings function. Press the "OK" to confirm; press "Left Button" to return to the previous menu interface.

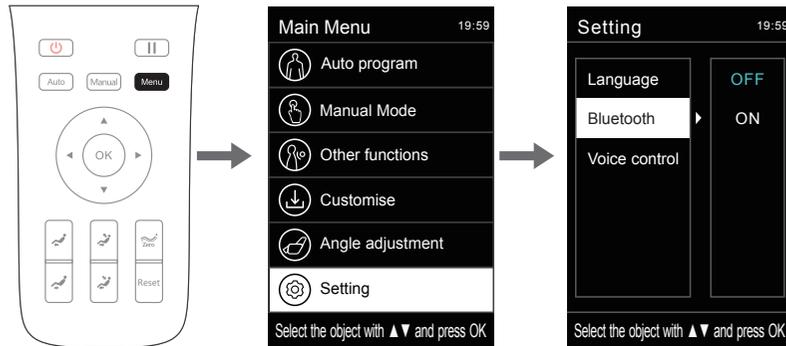

17) Language settings

- Language settings: including Vietnamese or English (the function has memory, that is, after such selection, power-off and restart, it will be the last selected state).
- Enter the settings interface. Press "Up Key" or "Down Key" to select the language settings. Press "Right Button" to enter the language settings. Press "OK" to select the required Simplified Vietnamese or English; press "Left Button" to return to the previous menu interface.

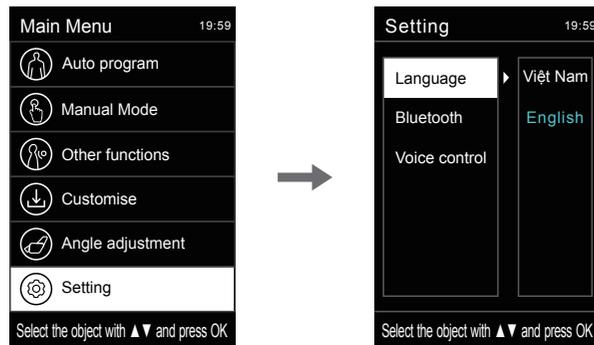

REMOTE CONTROL INSTRUCTIONS

1.Display:

- TFT color LCD showing the massage functions in the running state.

2.Power:

- Turn on or off the massage.

3.AUTO key:

- Enter the automatic function mode selection interface. For more information about the functions, refer to Page 12 of the Operation Instructions.

4.Manual key:

- Enter the manual function mode selection interface. For more information about the functions, refer to Page 13 of the Operation Instructions.

5.Up key:

- Up navigation key: select an object upward in the menu, or adjust the massage position upward.

6.OK:

- Confirm the selected massage function for massage.

7.Left key:

- Left navigation key: return to the previous menu, or select an object leftwards.

8.Down key:

- Down navigation key: select an object downward in the menu, or adjust the massage position downward.

9.Linkage sitting key:

- According to the required massage angle, simultaneously adjust linkage sitting function of the backrest and leg rest for the most comfortable linkage sitting massage posture.

10.Legrest Up key:

- Adjust the leg rest upward according to your own massage needs for the most comfortable massage posture.
- 11.Legrest down key:
- Adjust the leg rest downward according to your own massage needs for the most comfortable massage posture.

REMOTE CONTROL INSTRUCTIONS

11. Legrest down key:

- Adjust the leg rest downward according to your own massage needs for the most comfortable massage posture.

12. Linkage down key:

- According to the required massage angle, simultaneously adjust linkage down function of the backrest and leg rest for the most comfortable linkage down massage posture.

13. Pause key:

- Pause all the massage functions currently in service. Press this button again to continue to run all the massage functions before such pause.

14. Menu key:

- Enter the main menu selection interface. For more information about the functions, refer to Page 12 of the Operation Instructions.

15. Right key:

- Right navigation key: enter the sub-menu, or select an object rightwards.

16. Zero-gravity key:

- Adjust the angle of the massage chair for zero gravity. There are three automatic reclining modes.


17. Reset key:

- Press this key makes the massage chair angle restore to the original state.

USAGE METHODS


14) Angle adjustment - backrest angle

- Backrest angle adjustment: lying adjustment of the backrest.
- Enter the angle adjustment interface. Press "Up Key" or "Down Key" to select the backrest angle function. Then press "Up Key" or "Down Key" to adjust the backrest lying angle to the desired massage angle; press "Left Button" to return to the previous menu interface

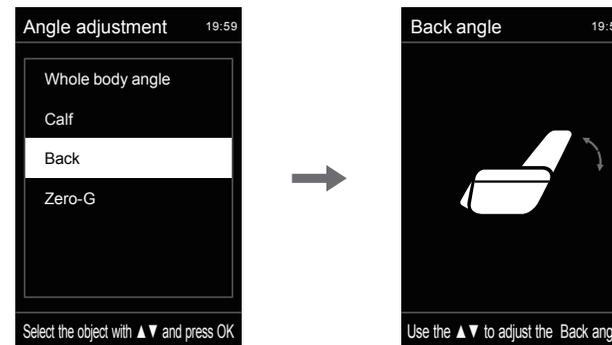

15) Angle adjustment - zero gravity


- Angle adjustment for zero gravity: There are three levels of angle adjustment for zero gravity and zero gravity OFF function. A total of four modes can be selected.
- Enter the angle adjustment interface. Press "Up Key" or "Down Key" to select the zero gravity function; press "OK" to enter the zero gravity interface, and then press "Up Key" or "Down Key" to select OFF, any of Level 1, Level 2 or Level 3 for the zero gravity angle. Then press "OK" to confirm; press "Left Button" to return to the previous menu interface.

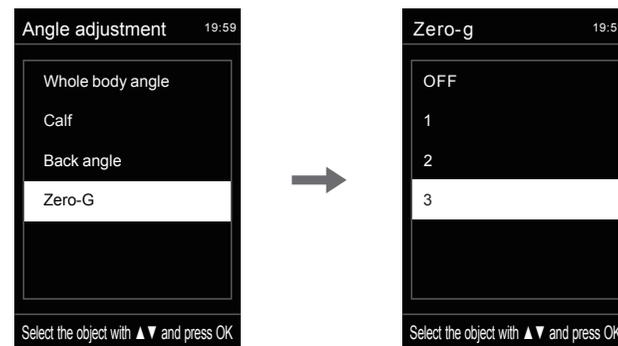

USAGE METHODS


12) Angle adjustment - angle for the whole body

- Angle adjustment for the whole body: adjust the angles of the backrest and the legrest at the same time.
- Enter the angle adjustment interface. Press "Up Key" or "Down Key" to select the angle function for the whole body. Then press "Up Key" or "Down Key" to adjust the angle for the whole body to the desired angle; press "Left Button" to return to the previous menu interface.

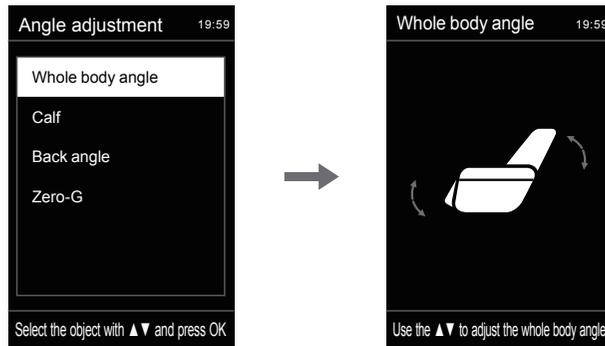


13) Angle adjustment - Leg angle

- Legrest angle adjustment: adjust the legrest upward and downward.
- Enter the angle adjustment interface. Press "Up Key" or "Down Key" to select the leg rest angle function. Then press "Up Key" or "Down Key" to adjust the legrest angle to the desired massage angle; press "Left Button" to return to the previous menu interface.

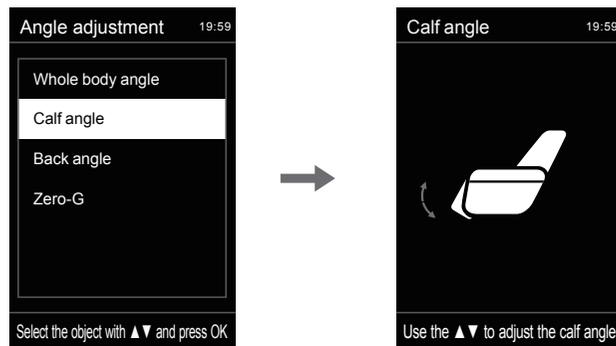

USAGE METHODS

I .Turn on the power supply of this product

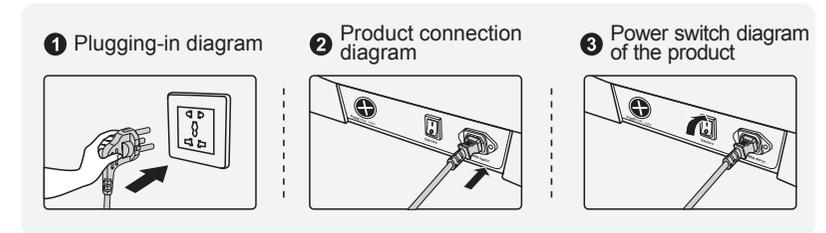

II. Massage function control operation

1.Standby

- After the massage chair is powered on and turned off, the remote controller is blank in the standby state (i.e. the LED lamp is off) and cannot perform any operation (except the key switch).




2.Start to massage (power switch)

- Press "Power" button to start massage, enter the boot screen, backrest maintains the default position.
- After automatically lying down to the default position, massage chair will automatically detect the shoulder position and body (note: fast massage mode only detect shoulder position, other modes detect whole body and shoulder position), waiting the detection finish. Or you can press "OK" button, the massage chair will directly start to massage in default without shoulder position and body sensing.




USAGE METHODS

- After testing the shoulder position, sounded with "di-di-di" notification tone, you can adjust the shoulder position by press up or down button according to users' height in small scale. There have 11 levels for shoulder adjustment. If without adjustment in 10 seconds, then into the massage function automatically. Or directly press "OK" button enter the massage immediately. If massage chair does not test shoulder position, the massage will start with default position.

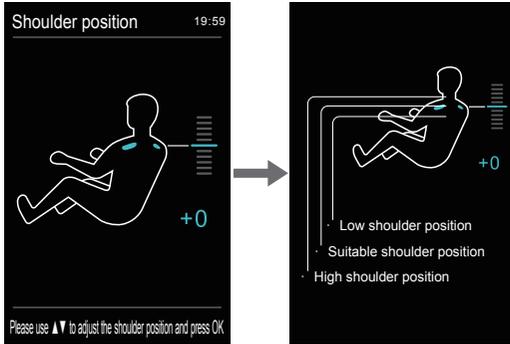


Button	Description
	Move the massage balls upward to a suitable shoulder position
	Move the massage balls downward to a suitable shoulder position

- If you need to adjust the shoulder position after the massage program is started, you can directly select "Shoulder Position Adjustment" to enter the shoulder position adjustment interface; you can press "Up Key" or "Down Key" to adjust the shoulder position height; you can press "Left Button" to return to the previous menu, or wait for 10 seconds to automatically return to the information interface.

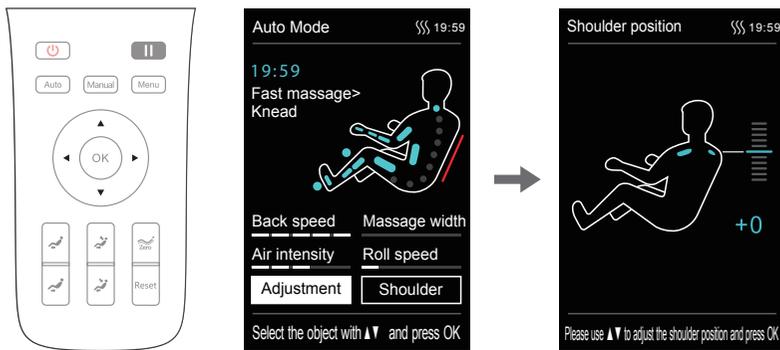


USAGE METHODS

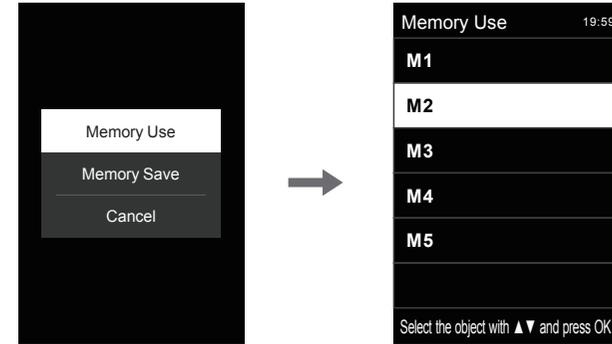


11) Angle adjustment (main menu-angle adjustment)


- Press "Menu Button" to enter the main menu interface. Press "Up Key" or "Down Key" to select the angle adjustment icon. Press "OK" to enter the angle adjustment interface.
- Angle adjustment includes: full body angle, backrest angle, legrest angle, zero gravity and other adjustment functions.
- After entering the angle adjustment interface, press "Up Key" or "Down Key" to select the desired angle adjustment function. After selecting any angle, press "OK" to confirm the selected angle adjustment. Press "Left Button" to return to the previous menu interface.

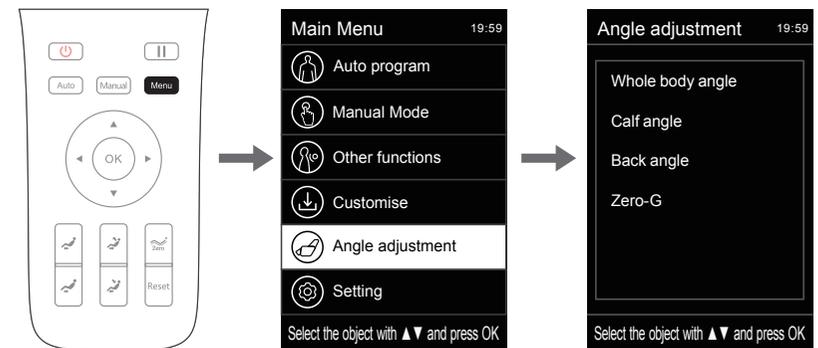

USAGE METHODS

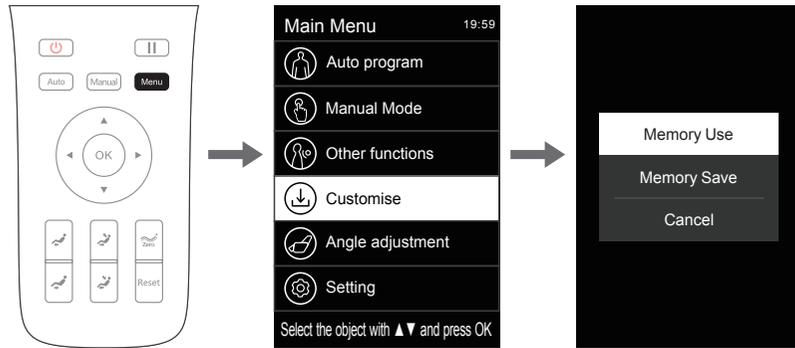**9) Intelligent storage - Save or Cancel**

- After entering the intelligent storage (saving) interface, press "Up Key" or "Down Key" to select the message function to be stored. Press "OK" to save or cancel.
- Enter the intelligent storage (saving) interface. Press "Up Key" or "Down Key" to select any one of the M1-M5 saving procedures. Select "OK" to confirm, and set up the saved message function; press "Left Button" to return to the previous menu interface.

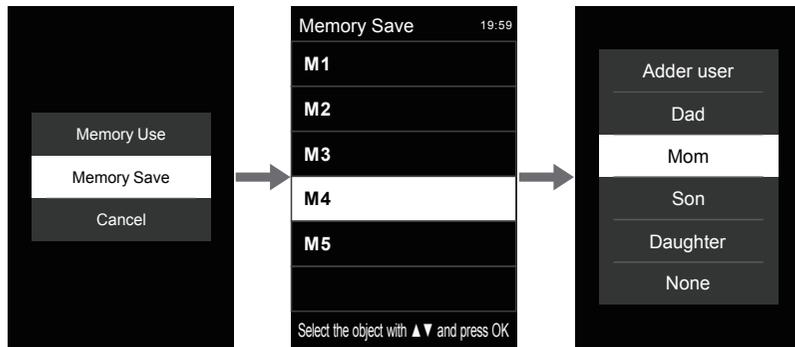

Note: Note: You can set up the saved message functions and techniques (e.g. father, mother, daughter, son, or none).


10) Intelligent storage - Call or Cancel

- After entering the intelligent storage (call) interface, press "Up Key" or "Down Key" to select the message function to be called. Press "OK" to enter the call or cancellation interface.
- Enter the intelligent storage (call) interface. Press "Up Key" or "Down Key" to select any one of the M1-M5 call procedures required to call the saved message program; press "Left Button" to return to the previous menu interface.


USAGE METHODS

3. Pause button

- While the message function is in service, press "Pause" button to temporarily stop all the message functions; press "Pause" button again to continue the message function before pause. If the pause time exceeds 20 minutes, all the message functions will be automatically turned off.

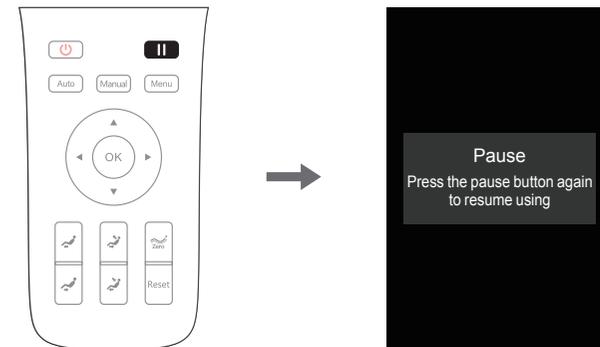**4. Menu button**


- Press "Menu Button" to enter the message function selection interface of the main menu, including automatic, manual, and other functions, intelligent storage, angle adjustment, settings and other function interfaces.
- Press "up" or "down" button choose the menu, press "OK" button to confirm enter the choosing menu interface. Press "Left" button to return the function selection information interface.

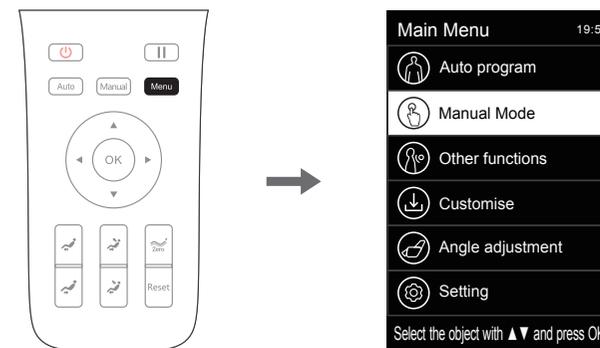

USAGE METHODS

1) Automatic function (main menu - automatic function)

- Press "Menu Button" to enter the main menu interface. Press "Up Key" or "Down Key" to select the automatic functions. Press "OK" (or press "AUTO Key" directly in the information interface) to enter the selection of automatic functions
- 12 special automatic massage functions are available: A1-Stroke prevent, A2-Comfort, A3-Relax, A4-Full body, A5-Neck and shoulder, A6-Stretch, A7-Back and waist, A8-Chiropractic, A9-Fatigue recovery, A10-Meridians, A11-Beautiful buttock, A12-Full airbag.
- Enter the automatic function interface. Press "Up Key" or "Down Key" to select any automatic function message program. It will automatically return to the information interface, which will be displayed as the automatic function information interface.

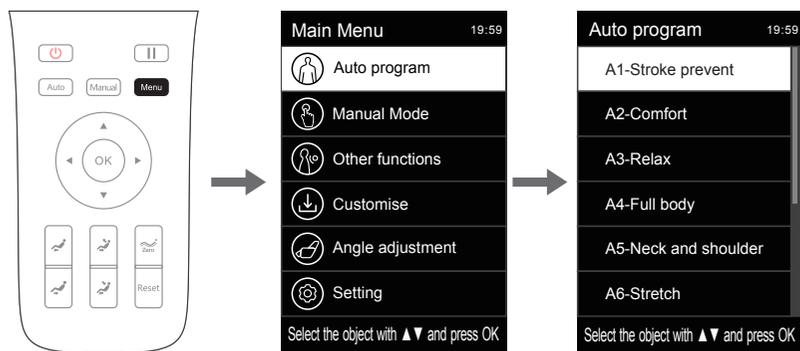


Function	Description
A1-Stroke prevent	Experience all the massage effects quickly. Blood circulation to the brain.
A2-Comfort	The most comfortable massage experience.
A3-Relax	Gentle and comfortable deep massage to relieve your body fatigue and relax the body.
A4-Full body	Deep massage to rejuvenate your body.

USAGE METHODS

7) Other functions - Calf kneading

- Calf kneading: enable or disable the calf kneading function.
- Enter another function interface. Press "Up Key" or "Down Key" to select the calf kneading function. Use "Right Button" to enable or disable such function. Then press "OK" to confirm. Press "Left Button" to return to the previous menu interface.

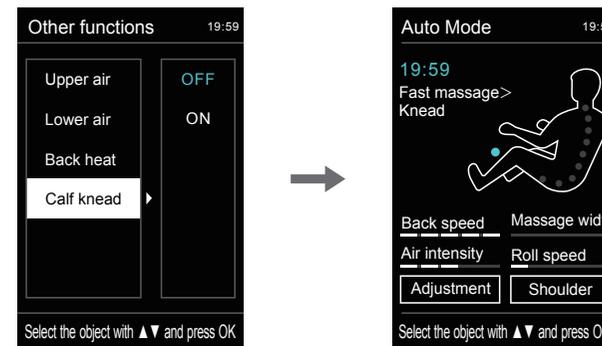


8) Intelligent storage (main menu - intelligent storage)

- Press the "Menu Button" to enter the menu interface. Press "Up Key" or "Down Key" to select the intelligent storage icon. Press "OK" to enter the intelligent storage interface, including: intelligent storage call, intelligent storage saving, cancellation and other storage functions.
- Intelligent storage call includes: M1, M2, M3, M4, and M5.
- Intelligent storage saving include: M1, M2, M3, M4, and M5.
- Call: Call the stored massage mode, massage technique, airbag mode and intensity under the current running status, the positions and angles of the legrest and the backrest, and other massage functions.
- Storage: Store the massage mode, massage technique, airbag mode and intensity under the current running status, the positions and angles of the legrest and the backrest, and other massage functions.
- Cancellation is the exit option.
- Note:** When saving the massage method, you can comment on the saved massage function and method (e.g. father, mother, son, daughter, or none).

USAGE METHODS


5) Other functions - airbag massage for the lower body

- Airbag massage for the lower body: including airbag massage functions for the legs and feet.
- Enter another function interface. Press "Up Key" or "Down Key" to select the airbag massage function for the lower body. Use "Right Button" to enable or disable the airbag massage function for the lower body. Then press "OK" to confirm. Press "Left Button" to return to the previous menu interface.

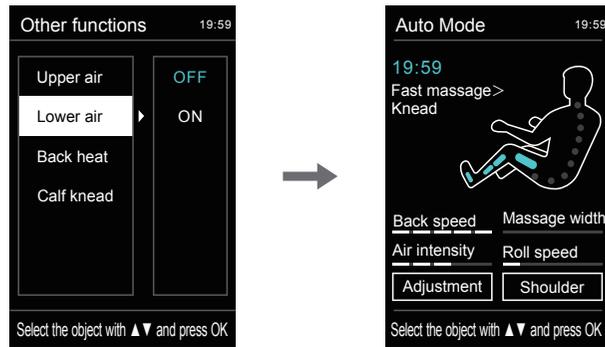

6) Other functions - back heating

- Back heating: enable or disable the back heating function; the back will become warmer after turning on heating for 3min.
- Enter another function interface. Press "Up Key" or "Down Key" to select the back heating function. Use "Right Button" to enable or disable such function. Then press "OK" to confirm. Press "Left Button" to return to the previous menu interface.

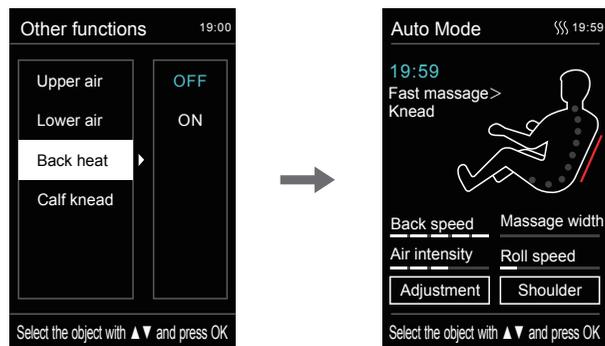

USAGE METHODS


A5-Neck and shoulder	Focus on the shoulder and neck massage.
A6-Stretch	Move your body parts up and down through the legrest and the backrest; airbag massage can be performed; and your legs can be.
A7-Back and waist	Focus on the back and waist massage.
A8-Chiropractic	Focus on chiropractic massage, and relax your body and mind.
A9-Fatigue recovery	Restore the full body from fatigue and pain; achieve relaxation, improve microcirculation, and quickly relieve fatigue for the back and limbs.
A10-Meridians	Stimulate the circulation of the blood and cause the muscles and joints to relax; quickly improve the body's microcirculation and enhance your immunity.
A11-Beautiful buttock	Effective massage on the buttocks and thighs to shape the body and reduce the accumulation of excess body fat.
A12-Full airbag	Focus on the full-body airbag massage to achieve full relaxation and deep massage of the body and fully relieve the fatigue and pain from your body.

Note: The automatic functions in service already selected will be displayed in the massage function status bar.

USAGE METHODS

2) Manual function (main menu - manual function)

- Press "Menu Button" to enter the main menu interface. Press "Up Key" or "Down Key" to select the manual function, and press "OK" to enter the manual function interface.
- 5 manual massage techniques/functions: Stop, Knead, Tap, knead & tap, Shiatsu, knock, etc.
- After entering the manual function interface, press "Up Key" or "Down Key" to select the required manual massage technique. Press "OK" to confirm. Press "Left Button" to return to the previous interface. After returning to the information interface, you will find that you have entered the information interface of the manual function.

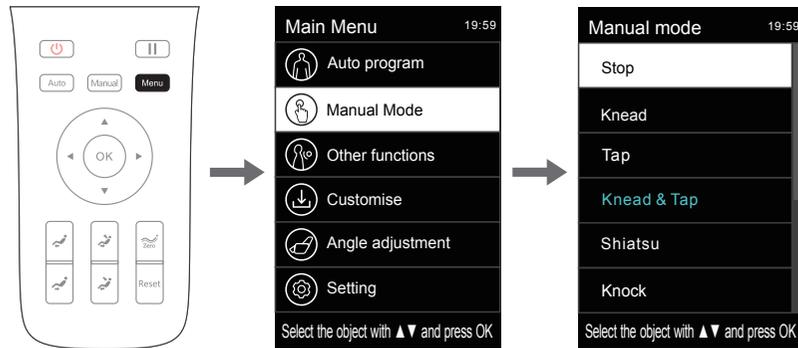


Function	Description	
Knead	Speed	· 5 speed levels adjustable · 3 width levels adjustable
Tap	Speed, width	
knead & Tap	Speed	
Shiatsu	Speed, width	
Knock	Speed, width	

USAGE METHODS

3) Other functions (main menu key - other functions)

- Press "Menu Button" to enter the main menu interface. Press "Up Key" or "Down Key" to select other function icons. Press "OK" to enter another function interfaces.
- Other functions include: airbag massage for the upper body/airbag massage for the lower body, back heating, leg kneading and other massage functions.
- After entering another function interface, press "Up Key" or "Down Key" to select the massage function. After selecting any one of the functions, use "Right Button" to enable or disable the function. Then press "OK" to confirm. Press "Left Button" to return to the previous menu interface.

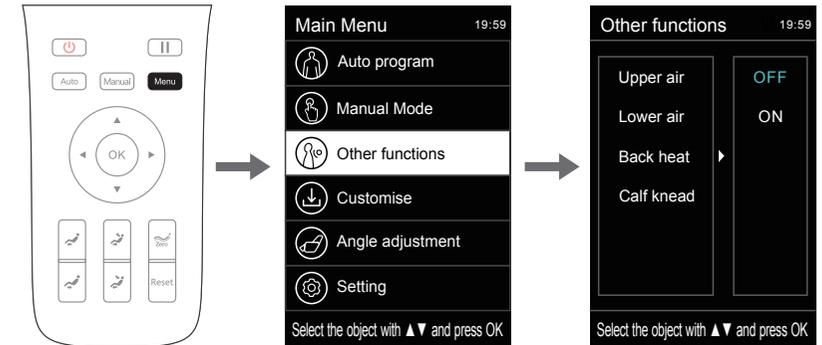

4) Other functions - airbag massage for the upper body

- Airbag massage for the upper body: including airbag massage for the upper arms pressure and airbag massage for the arms.
- Enter another function interface. Press "Up Key" or "Down Key" to select the airbag massage function for the upper body. Use "Right Button" to enable or disable the airbag massage function for the upper body. Then press "OK" to confirm. Press "Left Button" to return to the previous menu interface.

